Instructions for use / Gebrauchsanweisung
Aldosterone Urine Supplementary kit

REF**MS U-5200**

96

IVD

1. INTENDED USE

The **Aldosterone Urine** Supplementary kit is a reagent set intended to be used for the manual preparation of urine samples to be used with

- *Aldosterone ELISA* (MS E-5200).

For *in vitro* diagnostic use. For laboratory professional use.

This device does not provide any clinical information. This is obtained by using the respective assays.

1.1 Scientific Validity

The steroid hormone aldosterone is a potent mineral corticoid that is produced by the zona glomerulosa of the adrenal cortex in the adrenal gland. The synthesis and release are controlled by the renin-angiotensin-aldosterone system (RAAS) ¹, as well as by plasma potassium concentration ², the pituitary peptide ACTH, and by the blood pressure via pressure sensitive baroreceptors in the vessel walls of nearly all large arteries of the body ³. Aldosterone in urine is present as the glucuronated metabolite to improve hydrophilicity. This bond needs to be inverted to allow the precise and true measurement of Aldosterone in urine.

2. PRINCIPLE OF THE FUNCTION

The *Aldosterone Urine* Supplementary kit releases the binding of glucuronic acid from the aldosterone molecule using the *Release Reagent* to allow precise and true measurements. The addition of *Neutralization Buffer* is then used to neutralize the solution in order to allow for the assay to work as intended.

3. WARNINGS AND PRECAUTIONS

- This device is for *in vitro* diagnostic use only. For laboratory professional use only.
- Before starting the usage, read the instructions for use completely and carefully. Use the valid version of instructions for use provided with the device. Be sure that everything is understood.
- **Before use, carefully read the Instructions for Use provided with the Aldosterone ELISA kit.**
- Do not mix or use components from kits with different lot numbers. The kits may have been shipped or stored under different conditions.
- Do not use reagents beyond expiry date as shown on the kit labels.
- Reagents of other manufacturers must not be used together with the reagents of this test kit.
- All reagents in this kit are clear liquids. Changes in its appearance may affect the performance of the test. In that case, contact the manufacturer.
- Microbial contamination of reagents or samples may give false results.
- Allow the reagents to reach room temperature (20 °C to 26 °C) before starting the test.
- All indicated volumes must be performed according to the protocol. Optimal test results are only obtained when using calibrated pipettes and microtiter plate readers.
- Use reservoirs only for single reagents. This especially applies to the substrate reservoirs. Using a reservoir for dispensing a substrate solution that had previously been used for the conjugate solution may turn solution coloured. Do not pour reagents back into original vials as reagent contamination may occur.

General precautions

- Follow good laboratory practice and safety guidelines.
- Never pipet by mouth and avoid contact of reagents and samples with skin and mucous membranes.
- Do not smoke, eat, drink, or apply cosmetics in areas where samples or kit reagents are handled.
- Wear lab coats and disposable latex gloves when handling samples and reagents and where necessary safety glasses.

Biohazard information

- All materials and samples of human or animal origin must be handled as if capable of transmitting infectious diseases.
- Handling must be done in accordance with the procedures defined by appropriate national biohazard and safety guideline or regulation. Waste must be discarded according to local rules and regulations.

Information to chemical hazards and hazard classification

- Some reagents contain preservatives in non-declarable concentrations. Nevertheless, in case of contact with eyes or skin, flush immediately with water.
- Chemicals and prepared or used reagents must be treated as hazardous waste according to the national safety guideline or regulation.
- This product does not contain substances which have carcinogenic, mutagenic or toxic for reproduction (CMR) properties.

The following kit component(s) is(are) classified as hazardous:

Release Reagent  Warning	Hazard statement(s): H290 May be corrosive to metals. Precautionary statement(s): P234 Keep only in original container. P390 Absorb spillage to prevent material damage.
--	--

For detailed information, please refer to the Safety Data Sheet, which is available upon request directly from the manufacturer.

4. MATERIALS

4.1 Materials provided with the kit

MS U-5226 **RELEASE-BUFF** **Release Reagent** – Ready to use

Contents: Contains 1 M HCl.

Avoid contact with Release Reagent. It may cause skin irritation.

Volume: 1 x 3 ml

Hazards identification:

H290 May be corrosive to metals.

MS U-5243 **NEUTR BUF** **Neutralization Buffer** – Ready to use

Contents: Contains Tris buffer, pH 8.5.

Volume: 1 x 3 ml

MS U-5241 **DILUENT** **Dilution Buffer** – Ready to use

Contents: Contains PBS.

Volume: 2 x 25 ml

1 x Instructions for Use

1 x Certificate of Analysis (CoA)

4.2 Materials required but not provided

- Calibrated variable precision micropipettes
- Refrigerator for storage at 2 °C – 8 °C
- Distilled water
- Timer

4.3 Storage and Stability of the Kit

Unopened kits and reagents as well as **opened reagents** must be stored at 2 °C to 8 °C.

Once opened, reagent vials must be closed tightly again.

	Storage Temperature	Stability
Unopened kits and unopened reagents	2 °C to 8 °C	Until the expiration date printed on the label. Do not use reagents beyond this date!

4.4 Reagent Preparation

Bring all reagents and required number of strips to room temperature (20 °C to 26 °C) prior to use.

4.5 Disposal of the Kit

The disposal of the kit and all used materials/reagents must be performed according to the national regulations. Special information for this product is given in the Safety Data Sheet, section 13.

4.6 Damaged Test Kits

In case of any damage to the test kit or components, the manufacturer must be informed in writing, at the latest one week after receiving the kit. Damaged single components must not be used for a test run. They have to be stored until a final solution has been found. After this, they must be disposed of according to the official regulations.

5. SAMPLE COLLECTION, STORAGE AND PREPARATION

The following sample material can be used in this test:

urine

Samples containing sodium azide should not be used in the assay.

Please refer to the instructions for Use of *Aldosterone ELISA* (MS E-5200).

6. ASSAY PROCEDURE

6.1 Procedural Notes

- All reagents and samples must be allowed to come to room temperature (20 °C to 26 °C) before use.
- All reagents must be mixed without foaming.
- Do not interchange caps of reagent vials to avoid cross-contamination.
- Once the test has been started, all steps must be completed without interruption and in the same sequence for each step.
- Before starting the assay, it is recommended that all reagents are ready, caps removed, etc. This will ensure equal elapsed time for each pipetting step without interruption.
- **Test performance using fully automated analysis devices:**
Automated test performance using fully automated, open-system analysis devices is possible. However, the combination must be validated by the user.

6.2 Test Procedure

Please refer to the IFU of *Aldosterone ELISA* (MS E-5200).

6.3 Calculation of Results

Please refer to the IFU of *Aldosterone ELISA* (MS E-5200).

7. REFERENCE VALUES

Please refer to the IFU of *Aldosterone ELISA* (MS E-5200).

8. QUALITY CONTROL

Good laboratory practice requires that controls be run with each standard curve. A statistically significant number of controls should be assayed to establish mean values and acceptable ranges to assure proper performance.

It is recommended to use control samples according to state and federal regulations. The use of control samples is advised to assure the day-to-day validity of results. Use controls at both normal and pathological levels.

The controls and the corresponding results of the Quality Control Laboratory are stated in the Certificate of Analyses (CoA) added to the kit. The values and ranges stated on the CoA always refer to the current kit lot and must be used for direct comparison of the results.

If available, it is also recommended to make use of national or international Quality Assessment programs in order to ensure the accuracy of the results.

Apply appropriate statistical methods for analyzing control values and trends. If the results of the assay do not agree with the established acceptable ranges of control materials, patient results should be considered invalid.

In this case, please check the following technical areas: Pipetting and timing devices; photometer, expiration dates of reagents, storage and incubation conditions, aspiration and washing methods.

After checking the above-mentioned items without finding any error contact your distributor or the manufacturer directly.

9. PERFORMANCE CHARACTERISTICS

Performance is solely depending on the performance of the respective device (*Aldosterone ELISA* (MS E-5200)).

10. LIMITATIONS OF THE PROCEDURE

Reliable and reproducible results will be obtained when the assay procedure is performed with a complete understanding of the instructions for use and with adherence to good laboratory practice.

Any improper handling of samples or modification of this test might influence the results.

11. LEGAL ASPECTS

11.1 Reliability of Results

The test must be performed exactly as per the manufacturer's instructions for use. Moreover, the user must strictly adhere to the rules of GLP (Good Laboratory Practice) or other applicable national standards and/or laws. This is especially relevant for the use of control reagents. It is important to always include, within the test procedure, a sufficient number of controls for validating the accuracy and precision of the test.

The test results are valid only if all controls are within the specified ranges and if all other test parameters are also within the given assay specifications. If there is any doubt or concern regarding a result, please contact the manufacturer.

11.2 Therapeutic Consequences

Therapeutic consequences should never be based on laboratory results alone even if all test results are in agreement with the items as stated under point 11.1. Any laboratory result is only a part of the total clinical picture of a patient.

Only in cases where the laboratory results are in acceptable agreement with the overall clinical picture of the patient should therapeutic consequences be derived.

The test result itself should never be the sole determinant for deriving any therapeutic consequences.

11.3 Liability

Any modification of the test kit and/or exchange or mixture of any components of different lots from one test kit to another could negatively affect the intended results and validity of the overall test. Such modification and/or exchanges invalidate any claim for replacement.

Claims submitted due to customer misinterpretation of laboratory results subject to point 11.2 are also invalid. Regardless, in the event of any claim, the manufacturer's liability is not to exceed the value of the test kit. Any damage caused to the test kit during transportation is not subject to the liability of the manufacturer.

11.4 Reporting of Serious Incident

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

Symbols:

	Storage temperature		Manufacturer		Contains sufficient for <n> tests
	Use-by date		Batch code		For in-vitro diagnostic use only!
	Consult instructions for use		Content		CE marking of conformity
	Caution		Catalogue number		Distributor
	Date of manufacture				

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